

# HOW TO PAIR

*a wireless Haelvoet handset*



PLEASE FOLLOW THESE INSTRUCTIONS

1



Stand next to the bed with the handset in your hand and the magnet.

2



**Pairing:** Press the button "kneerest low" and hold the magnet against the handset for 3 seconds. You will hear a buzzer.

3



Release the button and the magnet and move closer to the control box until the buzzer frequency changes from slow to fast.

4



Confirm pairing by pushing once again the "kneerest low" button.

5



Double beep - Pairing OK

## FAQ

How much handsets / bed can be paired?

**You can only pair one wireless handset / bed.**


How long does the pairing between bed and handset last?

**The pairing will last until you pair a new handset.**

## WATCH THE DEMO VIDEO


Scan the QR code and learn how to pair a wireless Haelvoet handset

<https://express.adobe.com/video/cBHPHcMvecVYO>

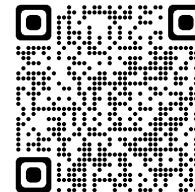

# HOW TO UNLOCK

Aron+ Junior bed

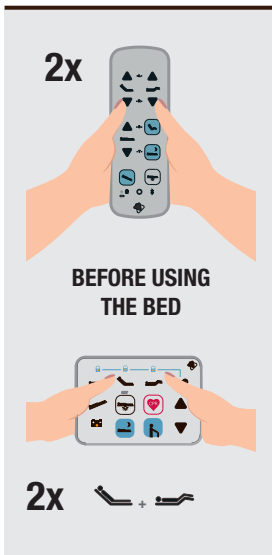

**1** Before you can use the bed, unlock the bed by pushing **2x the RESET buttons.**



**2** When the handset is activated, the LED at the bottom will flash **3x.**



**3** The handset can be used for **30 seconds** after the last key press.

Scan the QR code and learn how to unlock the wireless handset and the ACC nurse box



<https://express.adobe.com/video/cBHPHcMvecVYQ>

## HANDSET

Backrest adjustment

Kneerest adjustment

High/low adjustment

Relax position

Sleep position

Reverse Trendelenburg

Under bed light

LED warning light



## FREQUENTLY USED KEY COMBINATIONS

	Total lock (hold for 3 sec.)
	RESET (hold for 3 sec.) and Unlock (2x)
	Pairing mode (hold for 3 sec.)
	Trendelenburg position



Always keep eye contact with the bed when using the wireless handset. Please stop immediately using the wireless handset when the bed is not responding.